

INSTRUCTIONS FOR USE

Accura Y Connector Pack

Y Connector Pack

Please read instructions carefully before use.

Types of product

See table1


Table 1 types of Y Connector Pack

Descriptions				
No.	Types	Y Connector	Guide wire torque	Guide wire introducer
1	IYC9F	Bypass with extended tube and three-way stopcock	Plastic buckle, white	Yes

Contents of Y Connector Pack

- 1. Y connector (Hemostasis valve)
- 2. Extended tube with three-way stopcock of Y connector
- 3. Guide wire torque
- 4. Guide wire introducer (see figure1)

Figure 1

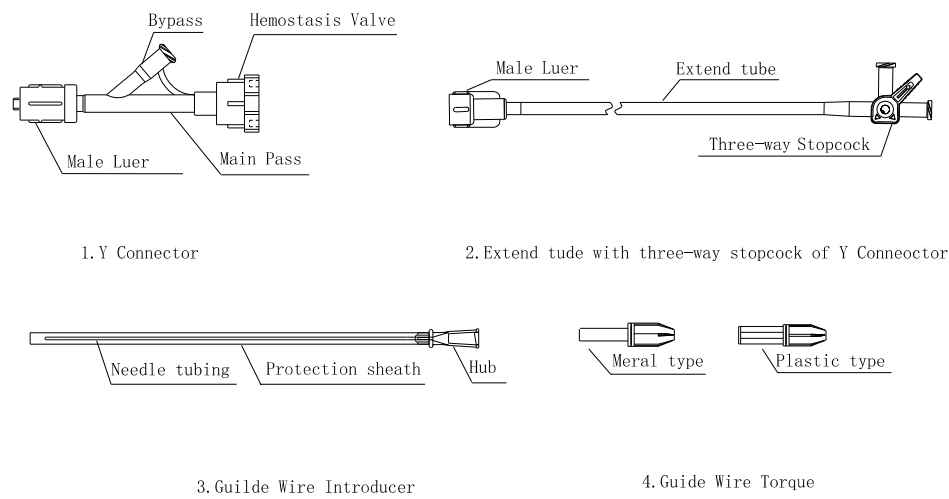

Indications

Y Connector intends to provide two tunnels into the blood vessel; this can introduce the contrast medium and guide wire or catheters during the interventional surgery.

Y Connector valve intends to prevent blood lost.

Guide wire introducer inserts the valve of Y connector, making a tunnel for guide wire to get through and protecting guide wire tip.

Guide wire torque holds guide wire and makes torque guide wire move easily.











Instructions for Use

- Check that the package is well sealed and without any damage.
- Connect the distal luer connector with catheter.
- Insert guide wire introducer into the valve on the proximal end of Y connector, and then tighten the valve.
- Insert guide wire through guide wire introducer into catheter.
- Remove guide wire introducer; screw close the valve of Y connector until no visible blood return.
- Using guide wire torque to push the guide wire is suggested.

Warnings

- Read the instructions before use.
- The device is designed for single use only. Do not desterilize and /or reuse.
- Do not use it after use before date (expiry date).
- Do not use it if the package is damaged or not well sealed.
- Luer connector must be well connected, and the valve must be closed. Incorrect connection, lose connection or the valve not being closed properly may cause leaking or air getting into blood vessel.
- Do not inject air into the blood vessel.
- Suggested working pressure is under 300 psi.
- For qualified physicians use only.
- Please dispose the device after use, according to the local legislation.

Signs and Legends

 Sterilized using ethylene oxide	 Lot number Batch number	 Manufacturer	 Do not re-use
 Use-by date	 Caution	 Qualified physicians use only	 Contents
 Catalogue Number	 Date of manufacture		

Storage

Store in a dry, dark, cool place.

The use before date (expiry date) see label.

Manufacturer**Accura Medizintechnik GmbH**

Max-Planck-Straße 33

611 84 Karben

Germany

**Distributor****CZ Pharma, s. r. o.**

Náměstí Smiřických 42

281 63 Kostelec nad Černými Lesy

Czech Republic

**Stav: 2013-05-24**